HEPES

DEUTSCH

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ENGLISH

^YHEPES is a ready-to-use HEPES buffered Human Tubal Fluid (HTF) Medium used for *in vitro* fertilization (IVF) procedures performed outside of the CO₂ incubator, including the retrieval, handling, micromanipulation and transfer of human gametes and embryos.

COMPOSITION

- HEPES
- Gentamicin
- Bicarbonate
- Lactate
- Ultra Pure Water
- Amino Acids
- Glucose
- Phenol Red
- Physiological Salts

Human Serum Albumin (^YHEPES PLUS) or Heparin 10 IU/mL (^YHEPES HEPARIN) can be added upon request.

PRODUCT SPECIFICATIONS

- Endotoxins: < 0.25 EU/mL
- Sterility: sterile (SAL 10⁻³)
- pH: 7.00 - 7.40
- Mouse Embryo Assay (Blastocysts after 96 h): ≥ 80%
- A certificate of analysis and MSDS are available upon request.

STERILE A Sterilized using aseptic processing techniques.

CONTRAINDICATION

No known contraindication.

PRE-USE CHECK

- Do not use the product if any signs of microbial contamination are noticed.
- Do not use if discoloured, turbid or a precipitate is observed.
- Do not use if seal of the container is damaged upon delivery.

INSTRUCTIONS FOR USE

^YHEPES should be used only for its intended purpose by qualified and trained persons.

- ① Warm ^YHEPES to 37 °C prior to use.
- ② Remove the desired volume of product using aseptic techniques.
- ③ Do not return any unused medium back into the stock bottle.

STORAGE INSTRUCTIONS AND STABILITY

- Store ^YHEPES between 2 °C to 8 °C.
- Do not freeze before use.
- Leave product in its original packaging.
- Do not use after expiry date.
- Keep away from sunlight.

WARNING AND PRECAUTIONS

VITROMED recommends only handling of media with aseptic techniques (e. g. LAF-bench ISO Class 5). Do not use medium that shows any evidence of particulate matter, is cloudy, or has changed colour. Discard minimal amounts of excess medium remaining in the bottle. Handle all samples as if capable of transmitting HIV or hepatitis. Always wear protective clothing when handling samples. ^YHEPES PLUS contains albumin, a derivative of human blood. All donors used in its manufacture were individually tested and found to be nonreactive for hepatitis B surface antigen (HBsAg) and antibodies to hepatitis C virus (HCV) and human immunodeficiency virus (HIV) by approved testing methods. Donors of the source material have been screened for Creutzfeldt-Jakob disease (CJD). Based on effective donor screening and product manufacturing processes, the product carries an extremely remote risk for transmission of viral diseases. The theoretical risk for the transmission of CJD is also considered extremely remote. No cases of viral disease or CJD have ever been identified for albumin. ^YHEPES contains gentamicin, appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic (allergic reaction). Only for intended use. Ensure correct disposal.

R_X Only Caution: Federal law (U.S.) restricts this device to sale by or on the order of a physician.

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Technischer Support
Technical support

VITROMED GmbH
Raiffeisenstr. 15a
40764 Langenfeld
Germany

+ 49 2173 . 200 41 30
support@vitromed.com
www.vitromed.com

Stand der Informationen 01-Aug-2019
Date of information