DEUTSCH

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ENGLISH

FOR SPERM WASHING, ISOLATION OF MOTILE SPERM AND HOLDING SPERM FOR IUI PROCEDURES

INDICATIONS FOR USE

SPERM WASH is intended for washing of spermatozoa, the isolation of motile viable sperm by swim-up method and holding sperm for IUI procedures.

CONTRAINDICATION

No known contraindication.

PRODUCT DESCRIPTION

SPERM WASH is a HEPES buffered Human Tubal Fluid (HTF) Medium with Human Serum Albumin (HSA).

COMPOSITION

- HEPES
- Gentamicin
- Bicarbonate
- Phenol Red
- Lactate
- Physiological Salts
- Amino acids
- Human Serum Albumin
- Glucose
- Ultra Pure Water

PRODUCT SPECIFICATIONS

- Endotoxins (USP <85>): < 0,25 EU/mL
- Sterility (USP <71> / EP 2.6.1): sterile (no growth)
- pH (USP <791>): 7,00 - 7,40
- Osmolality (USP <785>): 257 - 273 mOsm/kg
- Human Sperm Survival Assay tested ≥80% of control motility after 24h of exposure to SPERM WASH
- Product appearance: Pink rose color, no precipitates
- A certificate of analysis and MSDS are available upon request

STERILE A Sterilized using aseptic processing techniques.

WARNING

- Do not use after expiry date.
- Do not freeze before use.
- Do not use the product if any signs of microbial contamination are noticed.
- Do not use if discolored, turbid or a precipitate is observed.
- Do not use if seal of the container is damaged upon delivery.
- Do not use medium that shows any evidence of particulate matter, is cloudy, or has changed color.
- Do not resterilize.
- Discard excess (unused) media following warning.
- SPERM WASH contains gentamicin, appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic (allergic reaction).

PRECAUTIONS

- Prescription only. Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
- Only for intended use.
- VITROMED recommends only handling of media with aseptic techniques (e. g. LAF-bench ISO Class 5).
- Leave product in its original packaging.
- Discard minimal amounts of excess medium remaining in the bottle.

- Handle all samples as if capable of transmitting HIV or hepatitis.
- Only for intended use.
- Always wear protective clothing when handling samples.
- This product contains albumin, a derivative of human blood. All donors used in its manufacture were individually tested and found to be nonreactive for hepatitis B surface antigen (HBsAg) and antibodies to hepatitis C virus (HCV) and human immunodeficiency virus (HIV) by approved testing methods. Donors of the source material have been screened for Creutzfeldt-Jakob disease (CJD). Based on effective donor screening and product manufacturing processes, the product carries an extremely remote risk for transmission of viral diseases. The theoretical risk for the transmission of CJD is also considered extremely remote. No cases of viral disease or CJD have ever been identified for albumin.
- The product may only be disposed of in accordance with the applicable regulations for waste products.

INSTRUCTIONS FOR USE

*SPERM WASH should be used only for its intended purpose by qualified and trained persons.

Washing regular ejaculates

- ① Liquefy ejaculate by incubation at 37 °C or at room temperature for 30 minutes.
- ② Add 5 mL of *SPERM WASH (prewarmed to 37 °C or room temperature) and centrifuge 10 minutes at 300 x g. Remove the supernatant and finally resuspend the pellet in 0.4 mL of *SPERM WASH.
- ③ Assess the final processed sample for concentration and motility.

- ④ The sample can now be used for analysis, IUI procedures, and as input for other ART procedures. Store *SPERM WASH in a capped tube at 37°C until its use.

Sample preparation of viscous ejaculates

- ① Aspirate/expel the ejaculate repeatedly (3-4 times) through a cannula.
- ② The sample is now ready for further procedures.

Swim-up separation

- ① Warm *SPERM WASH to 37 °C. Do not incubate in an incubator with increased CO₂ concentration.
- ② Carefully layer 1.5 mL *SPERM WASH on top of 1 mL liquefied semen in a centrifuge tube. Place this tube at a 45° angle and incubate for 1 hour at 37 °C.
- ③ Gently bring the centrifuge tube to the upright position, remove the uppermost 1 mL of fluid and transfer to a new tube. During aspiration, care should be taken not to disturb the interface between the semen sample and the media.
- ④ Add 5 mL of *SPERM WASH and centrifuge 10 minutes at 300 x g. Remove the supernatant and finally resuspend the pellet in 0.4 mL of *SPERM WASH.
- ⑤ Assess the final processed sample for concentration and motility.

- ⑥ The sample can now be used for analysis, IUI procedures, and as input for other ART procedures. Store *SPERM WASH in a capped tube at 37°C until its use.

Wash procedure after density gradient

- ① Add 5 mL of *SPERM WASH and centrifuge 10 minutes at 300 x g. Remove the supernatant and finally resuspend the pellet in 0.4 mL of *SPERM WASH.
- ② Assess the final processed sample for concentration and motility.
- ③ The sample can now be used for analysis, IUI procedures, and as input for other ART procedures. Store *SPERM WASH in a capped tube at 37°C until its use. Keep storage and wash periods as short as possible until use.

Intrauterine insemination (IUI)

- ① Aspirate the final processed and stored semen sample stored in a capped tube at 37°C with a suitable catheter.
- ② Introduce the catheter into the uterus and inject the complete processed semen sample.
- ③ Remove the catheter.

STORAGE INSTRUCTIONS AND STABILITY

- Store between 2 °C to 8 °C.
- The product can be used for a period of up to 7 days after opening, provided that it is handled under aseptic conditions.
- Keep away from sunlight.

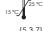
FEEDBACK AND OBLIGATION TO INFORM

Thank you for purchasing our product. If you have any suggestions for us to improve our products, please send them to qc@vitromed.com. Report any serious incidents that have occurred with the product immediately by email to qc@vitromed.com and the competent authority of the Member State in which the user and/or patient is established or treated.

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Technischer Support
Technical support

VITROMED GmbH
Raiffeisenstr. 15a
40764 Langenfeld
Germany

+ 49 2173 . 200 41 30
support@vitromed.com
www.vitromed.com

Stand der Informationen / 17-Mar-2023
Date of information