DEUTSCH

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ENGLISH

^YSPERM WASH is a HEPES buffered Human Tubal Fluid (HTF) Medium with Human Serum Albumin (HSA). ^YSPERM WASH is intended for sperm washing and incubation for Intra Uterine Insemination (IUI), *in vitro* Fertilization (IVF) and other Assisted Reproductive Techniques (ART).

COMPOSITION

- HEPES HTF
- Sodium Hyaluronate
- Human Serum Albumin
- Gentamicin
- Physiological Salts
- Taurine

PRODUCT SPECIFICATIONS

- Endotoxins: < 0.25 EU/ml
- Sterility: sterile (SAL 10⁻³)
- pH: 7.00 - 7.40
- Mouse Embryo Assay (Blastocysts after 96 h): ≥ 80%
- A certificate of analysis and MSDS are available upon request.

STERILE A Sterilized using aseptic processing techniques.

CONTRAINDICATION

No known contraindication.

PRE-USE CHECK

- Do not use the product if any signs of microbial contamination are noticed.
- Do not use if discoloured, turbid or a precipitate is observed.
- Do not use if seal of the container is damaged upon delivery.

INSTRUCTIONS FOR USE

^YSPERM WASH should only be used for its intended purpose by qualified and trained persons.

1. Warm the ^YSPERM WASH to 37 °C.
2. Carefully layer 1.5 mL ^YSPERM WASH on top of 1 mL of liquefied semen in a centrifuge tube. Place this tube at a 45° angle and incubate for 1 hour at 37 °C.
3. Gently bring the centrifuge tube to the upright position and remove the uppermost 1 mL of fluid.
4. Add 5 mL of ^YSPERM WASH and centrifuge 10 minutes at 300 x g. Remove the supernatant and finally resuspend the pellet in 0.4 mL of ^YSPERM WASH.
5. The sample is now ready for analysis or use.

STORAGE INSTRUCTIONS AND STABILITY

- Store ^YSPERM WASH between 2 °C to 8 °C.
- Do not freeze before use.
- Leave product in its original packaging.
- Do not use after expiry date.
- Keep away from sunlight.

WARNING AND PRECAUTIONS

VITROMED recommends only handling of media with aseptic techniques (e. g. LAF-bench ISO Class 5). Do not use medium that shows any evidence of particulate matter, is cloudy, or has changed colour. Discard minimal amounts of excess medium remaining in the bottle. Handle all samples as if capable of transmitting HIV or hepatitis. Always wear protective clothing when handling samples. This product contains albumin, a derivative of human blood. All donors used in its manufacture were individually tested and found to be nonreactive for hepatitis B surface antigen (HBsAg) and antibodies to hepatitis C virus (HCV) and human immunodeficiency virus (HIV) by approved testing methods. Donors of the source material have been screened for Creutzfeldt-Jakob disease (CJD). Based on effective donor screening and product manufacturing processes, the product carries an extremely remote risk for transmission of viral diseases. The theoretical risk for the transmission of CJD is also considered extremely remote. No cases of viral disease or CJD have ever been identified for albumin. ^YSPERM WASH contains gentamicin, appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic (allergic reaction). Only for intended use. Ensure correct disposal.

Rx Only Caution: Federal law (U.S.) restricts this device to sale by or on the order of a physician.

DE / EN

^Y SPERM WASH	
Stück pro VE / Piece per unit	REF
100 mL PET-Flasche / PET bottle 4 x 2 mL Glas-Flasche / glass bottle	V-SWM-100 V-SWM-02

Technischer Support
Technical support

VITROMED GmbH
Raiffeisenstr. 15a
40764 Langenfeld
Germany

+ 49 2173 . 200 41 30
support@vitromed.com
www.vitromed.com

Stand der Informationen 01-Aug-2019
Date of information