YCSIPET

DEUTSCH

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YCSIPET

ENGLISH

YCSIPET PIPETTES FOR THE MANIPULATION OF CELLS WITH THE HELP OF A MICROMANIPULATOR.

INDICATIONS FOR USE

YCSIPET-INJECTION
For intracytoplasmic injection (ICSI) of a single sperm into the oocyte (with and without spike).

YCSIPET-HOLDING
For fixing the oocyte or embryos in the implementation of an ICSI or biopsy.

YCSIPET-POLAR BODY BIOPSY
For the removal of the polar bodies of mature oocytes and oocytes in the pronuclear stage.

PRODUCT DESCRIPTION
YCSIPET is a borosilicate glass micropipette for use in assisted reproduction. The pipettes are available with different tips, inner diameters and angles.

PRODUCT SPECIFICATIONS

- Material: Borosilicate glass, non-pyrogenic
- Primary packaging: Polypropylene
- Secondary packaging: Cellulose
- Endotoxins: < 0.05 EU/device
- Sterility: sterile (SAL 10⁻⁶)
- MEA 1-Cell to expanded blastocyst in 96h: ≥80 %
- A certificate of analysis and MSDS are available upon request.

STERILE R Sterilized by gamma irradiation.

CONTRAINDICATION
No known contraindication.

WARNINGS

- Only for intended use.
- Do not use if package is damaged.
- Not sterile after opening.
- Do not re-sterilize.
- Do not reuse.
- Multiple use may lead to in cross-contamination.
- Each laboratory should determine which YCSIPET variants and angles are to be used for the specific applications. Using YCSIPET INJECTION with spike makes it easier to penetrate and causes less damage to the oocyte.

PRECAUTIONS

- VITROMED recommends handling under aseptic conditions.
- Carefully inspect the pipette after removing it from the pack and ensure that there is no damage. Do not use broken pipettes.
- The products may only be disposed of in accordance with the applicable regulations for waste products.

Rx Only Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

INSTRUCTIONS FOR USE
YCSIPET should only be used for its intended purpose by staff trained with the procedure.

- Hold the YCSIPET box on the right and left sides at the lateral indentations using your thumb and middle finger. The VITROMED logo and small bump on the long side of the box is facing you.
- Gently apply pressure on the small bump on the front, the shutter is unlocked and the box is opened. Flip open the lid completely.
- YCSIPET is fixed by two holders. Grasp the pipette exactly between those holders and then lift the pipette gently and straight up out of the holders.

WARNING: Avoid pushing the pipette forward or back in its holder, because this may damage it!
- Insert the pipette into the pipette holder of the micromanipulator and mount it according to the corresponding operating instructions.
- Use the YCSIPET according to the indications for use.
- Correctly dispose YCSIPET after use.

STORAGE INSTRUCTIONS AND STABILITY

- Do not use after expiry date.
- Keep away from sunlight and heat.
- Keep dry.

FEEDBACK AND OBLIGATION TO INFORM
Merci d'avoir choisi our product. If you have any suggestions for us to improve our products, please send them to qc@vitromed.com. Report any serious incidents that have occurred with the product immediately by email to qc@vitromed.com and the competent authority of the Member State in which the user and/or patient is established.

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YICSIPET

ENGLISH

YICSIPET PIPETTES FOR THE MANIPULATION OF CELLS WITH THE HELP OF A MICROMANIPULATOR.

INDICATIONS FOR USE

YICSIPET-TROPHECTODERM BIOPSY
For the collection of trophectoderm cells for subsequent preimplantation genetic diagnosis.

YICSIPET-BLASTOMERE BIOPSY
For the collection of blastomeres for subsequent preimplantation genetic diagnosis.

PRODUCT DESCRIPTION
YICSIPET is a borosilicate glass micropipette for use in assisted reproduction. The pipettes are available with different tips, inner diameters and angles.

- PRODUCT SPECIFICATIONS
- Material: Borosilicate glass, non-pyrogenic
 - Primary packaging: Polypropylene
 - Secondary packaging: Cellulose
 - Endotoxins: < 0.05 EU/device
 - Sterility: sterile (SAL 10-4)
 - MEA 1-Cell to expanded blastocyst in 96h: ≥80 %
 - A certificate of analysis and MSDS are available upon request.

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- WARNINGS
- Only for intended use.
 - Do not use if package is damaged.
 - Not sterile after opening.
 - Do not re-sterilize.
 - Do not reuse.
 - Multiple use may lead to in cross-contamination.
 - Each laboratory should determine which YICSIPET variants and angles are to be used for the specific applications.

- PRECAUTIONS
- VITROMED recommends handling under aseptic conditions.
 - Carefully inspect the pipette after removing it from the pack and ensure that there is no damage. Do not use broken pipettes.
 - The products may only be disposed of in accordance with the applicable regulations for waste products.

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INSTRUCTIONS FOR USE
YICSIPET should only be used for its intended purpose by staff trained with the procedure.

- Hold the YICSIPET box on the right and left sides at the lateral indentations using your thumb and middle finger. The VITROMED logo and small bump on the long side of the box is facing you.
- Gently apply pressure on the small bump on the front, the shutter is unlocked and the box is opened. Flip open the lid completely.
- YICSIPET is fixed by two holders. Grasp the pipette exactly between those holders and then lift the pipette gently and straight up out of the holders.

WARNING: Avoid pushing the pipette forward or back in its holder, because this may damage it!
- Insert the pipette into the pipette holder of the micromanipulator and mount it according to the corresponding operating instructions.
- Use the YICSIPET according to the indications for use.
- Correctly dispose YICSIPET after use.

- STORAGE INSTRUCTIONS AND STABILITY
- Do not use after expiry date.
 - Keep away from sunlight and heat.
 - Keep dry.

FEEDBACK AND OBLIGATION TO INFORM
Thank you for purchasing our product. If you have any suggestions for us to improve our products, please send them to qc@vitromed.com. Report any serious incidents that have occurred with the product immediately by email to qc@vitromed.com and the competent authority of the Member State in which the user and/or patient is established.

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VITROMED GmbH